


# TempTray™

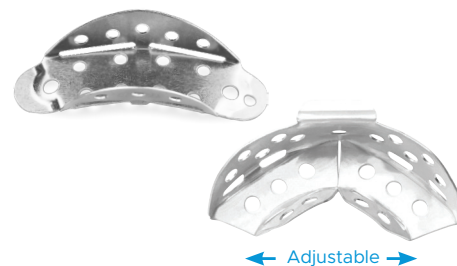

← Adjustable →

**TempTrays™** are designed for accuracy and versatility. Made from rigid aluminum, TempTrays are customizable and easy to use for both anterior and posterior temporary restorations. TempTrays can be easily adjusted for malaligned teeth and fit virtually all arch shapes. With high, perforated walls, the TempTray is designed to firmly retain the matrix material while eliminating the chance of distortion. TempTrays are the perfect companion to Template® Ultra Quick Matrix Material and are a fraction of the cost of quadrant or dual-arch trays.


## Posterior TempTray:

- Multiple teeth can be captured.
- Posterior tabs for easy placement. Can be folded in to prevent retromolar pad impingement.
- Adjustable, perforated sidewalls.
- Single use. Recyclable.


## Anterior TempTray:

- Unique slot arch design allows for adjustment to fit all arch widths.
- Adjustable handle for easy placement and removal.
- High sidewall to capture any tooth length.
- Perforated sidewall.
- Single use. Recyclable.

*Dentistry and photography courtesy of Bob Margeas, DDS.*

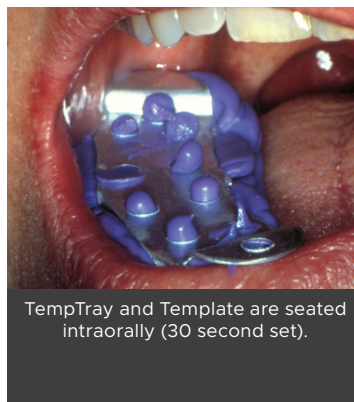


TempTray and Template are seated intraorally (30 second set).

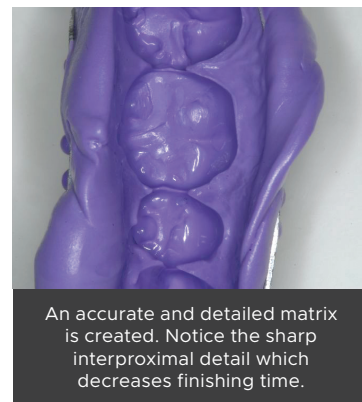

An accurate and detailed matrix is created. Notice the sharp interproximal detail which decreases finishing time.

## ANTERIOR PROVISIONAL BRIDGE CLINICAL TECHNIQUE

*Dentistry and photography courtesy of Robert Lowe, DDS.*

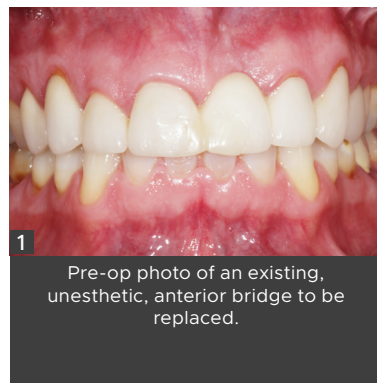


**1** Pre-op photo of an existing, unesthetic, anterior bridge to be replaced.



**2** Load the customizable Anterior TempTray with Template Matrix Material and place intraorally within the 30-second working time.




**3** After only 30 seconds, remove the tray from the mouth. The result is an accurate and detailed matrix.

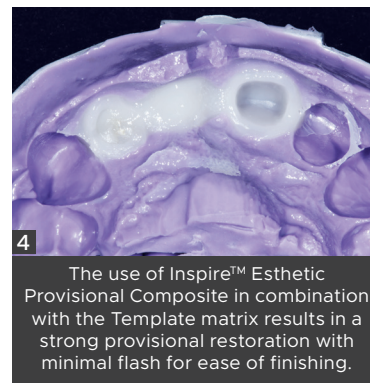

**4** The use of Inspire™ Esthetic Provisional Composite in combination with the Template matrix results in a strong provisional restoration with minimal flash for ease of finishing.



**5** Line the Inspire provisional with Cling2® Temporary Cement and seat on the prepared teeth within the 30-second working time. After 60-90 seconds, the excess cement is easily removed.

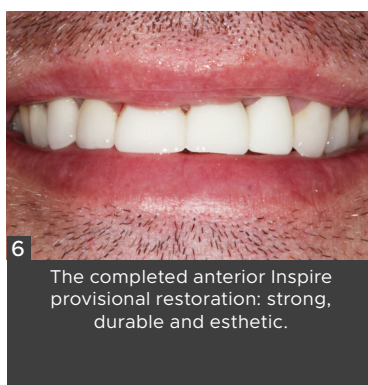

**6** The completed anterior Inspire provisional restoration: strong, durable and esthetic.

**TEMPTRAY 50-PACK (046050)**

**TEMPTRAY 100-PACK (046100)**

**TEMPTRAY 250-PACK (046102)**

**TEMPTRAY TRIAL KIT (046152)**

Contains: 50 x TempTrays, 50 x Anterior TempTrays

**ANTERIOR TEMPTRAY 50-PACK (046150)**

**ANTERIOR TEMPTRAY 100-PACK (046151)**

To view the latest news, education and more, visit:

[www.clinicianschoice.com](http://www.clinicianschoice.com)



 **Clinician's Choice®**  
Designed for better dentistry.