

# Wall Mounting Bracket Installation Instructions

Part Number: 2170100100

VistaScan  
Ultra View  
Image Plate Scanner




## Installation Instructions

**Scope of Delivery.** The following items are included in the scope of delivery (possible variations due to country-specific requirements and/or import regulations):

- Wall mounting bracket (Qty 1) . . . . 2141-001-01
- Knurled screws (Qty 2) . . . . . 34949
- Washers, #6 (Qty 2) . . . . . 30051
- Polyurethane bumpers (Qty 2) . . . . 34950

### Install the Wall Mounting Bracket as follows:

1. Align the Wall Mount Face of the wall bracket at a convenient location on the wall.
2. Using the bracket face as a template, mark screw hole locations as necessary.
3. Secure the bracket to the wall using three suitable fasteners for the wall material.
4. Lay the scanner on its back and install the two Polyurethane Bumpers on the foot (see image).
5. Thread the cable through the cable duct.
6. Place the scanner on the wall bracket Scanner Mount Platform, connect the cables, and secure with two Knurled screws and washers supplied.

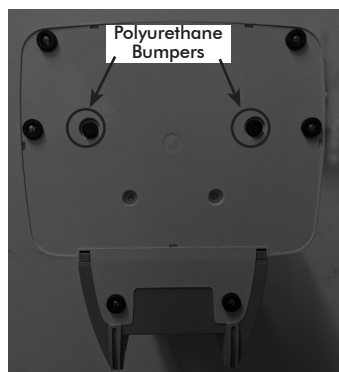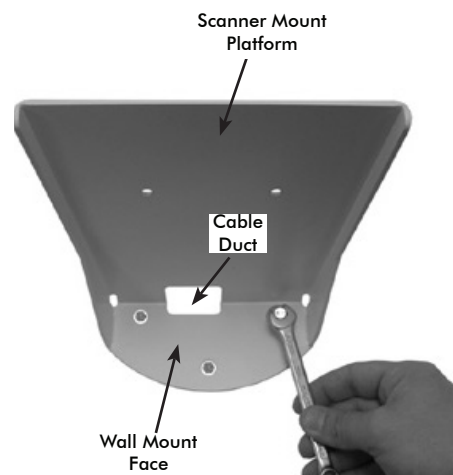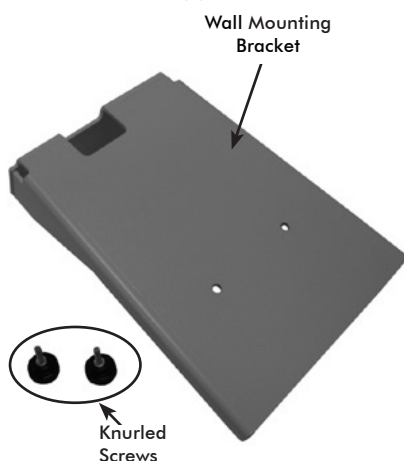

Manufactured for / Distributed by:

DÜRR DENTAL SE | Höpfigheimer Str. 17 | 74321 Bietigheim-Bissingen | Germany

Fon: +49 7142 705-0 | [www.duerddental.com](http://www.duerddental.com) | [info@duerrdental.com](mailto:info@duerrdental.com)

2170100033L02



Air Techniques, Inc. | 1295 Walt Whitman Road | Melville, New York 11747- 3062, USA  
Air Techniques, a Dürr Dental Company

