

# NEW PIEZON® PI MAX INSTRUMENT


## FOR GENTLE AND EFFECTIVE CALCULUS REMOVAL AROUND IMPLANTS, RESTORATIONS AND ORTHODONTIC APPLIANCES

The new PIEZON® PI MAX Instrument is the latest innovation from the EMS Research Center in Nyon, Switzerland. Developed to maximize patient comfort together with the minimally invasive Guided Biofilm Therapy protocol, its high-tech PEEK material provides 50% more flexibility and accessibility than the previous generation.



### MATERIALS

- ▶ Instrument tip: PEEK with 30% carbon fiber
- ▶ Tip holder: Medical grade stainless steel

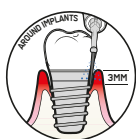

### CLINICAL APPLICATIONS

- ▶ Non-surgical treatments around implants, restorations and orthodontic appliances



### COMPATIBILITY

- ▶ AIRFLOW® Prophylaxis Master
- ▶ EMS Devices equipped with PIEZON NO PAIN® technology



### DEVICE SETTINGS

- ▶ Power: Start at 30%, increase if needed up to 50%
- ▶ Water: Above 70% at all times



### LIFETIME

- ▶ Instrument tip: Up to 3mm wear or 20 treatments
- ▶ The PIEZON® PI MAX Instrument Tool helps to check the wear of the instrument tip & replace it when needed
- ▶ Tip holder: 1,000 reprocessing cycles



WATCH  
VIDEO ON  
THE PI MAX

Available in Australia from June 1st 2023. For more details, contact your GBT Expert or EMS Distributor.

REF	DESCRIPTION	CONFIGURATION
DS-010A/A	PIEZON® PI MAX Instrument	1 PI MAX Instrument with its dedicated combitorque
DT-065A/A	PI MAX Instrument Tip (x4)	4 PI Instrument tips in a green packaging
FV-117	PI MAX Instrument Tool	1 PI Tool to exchange instrument tip and check the wear

**TRY THE PI MAX INSTRUMENT TODAY**

**EMS**<sup>+</sup>  
MAKE ME SMILE.