

Mounting Instructions - 'Clip-it'

1. Choose any smooth surface with enough room to mount this brochure and the 'Clip-it' supplied in the immediate area of your ultrasonic cleaner.
2. Clean the area with methylated spirits and then wipe with a clean dry cloth.
3. Peel the backing paper from the 'Clip-it' making sure not to touch the adhesive.
4. Press 'Clip-it' firmly into vertical position and hold for 30 seconds, then leave for 24 hours before using.
5. Mount this brochure with 'Directions for Use' visible.

WARNING Do not use 'Clip-it' on wallpaper, porous or rough surfaces, or on a surface where the temperature exceeds 60°C.

All instructions follow

AS 2773.1 - 1998

Ultrasonic cleaners for health care facilities - Non-portable.

AS 2773.2 - 1999

Ultrasonic cleaners for health care facilities - Benchtop.

AS 4187 - 2014

Cleaning, disinfecting and sterising reusable medical and surgical instruments and equipment, and maintenance of associated environments in health care facilities.

The user must follow the appropriate standards (ISO/AS/NZ) when using this product

AS 2773.1 - 1998

Ultrasonic cleaners for health care facilities - Non-portable.

AS 2773.2 - 1999

Ultrasonic cleaners for health care facilities - Benchtop.

AS 4187 - 2014

Cleaning, disinfecting and sterising reusable medical and surgical instruments and equipment, and maintenance of associated environments in health care facilities.

MEDiDENT®

www.medident.com.au



Hendrick's

Instruments Services

ULTRASONITEST™



Re-order from

MEDiDENT®

Online: www.medident.com.au

Phone: 1300 886 674

Email: medident@medident.com.au

PERFORM TEST DAILY

AS 4187 – 1988 Section 2.10.2.3 & 7.1

Testing must be carried out in accordance with the following instructions and the Ultrasonic Cleaner Manufacturer's Directions for use.

DIRECTIONS FOR USE

1. Commence with a clean, empty ultrasonic machine.
2. Fill with water to the recommended level.
3. Perform 'Degassing' procedure as per manufacturers instructions.
4. Rub the surface of the disc with the special test pencil provided.
5. Immerse the disk in the centre of the ultrasonic cleaning tank on the rack provided by the manufacturer.
6. Set the time for 3 minutes and start the ultrasonic machine.

WARNING

Do not touch when ultrasonic machine is operating.

7. When cycle is complete, remove ULTRASONITEST.

NOTE

The ultrasonic cleaner should completely remove the pencil marks within 3 minutes.

8. Record results on Results Sheet.
9. Check disk for surface integrity, if smooth, replace the disk (up to 60 uses per disk).
10. Return disk, test pencil and result sheet to 'Clip-it' immediately after use.

Re-order from

MEDiDENT®

Online: www.medident.com.au

Phone: 1300 886 674

Email: medident@medident.com.au

RESULTS SHEET - ULTRASONITEST

No.	Date	Pass	Fail	Signature
1				
2				
3				
4				
5				
6				
7				
8				
9				
10				
11				
12				
13				
14				
15				
16				
17				
18				
19				
20				
21				
22				
23				
24				
25				
26				
27				
28				
29				
30				

No.	Date	Pass	Fail	Signature
31				
32				
33				
34				
35				
36				
37				
38				
39				
40				
41				
42				
43				
44				
45				
46				
47				
48				
49				
50				
51				
52				
53				
54				
55				
56				
57				
58				
59				
60				