


the future in parallel post systems

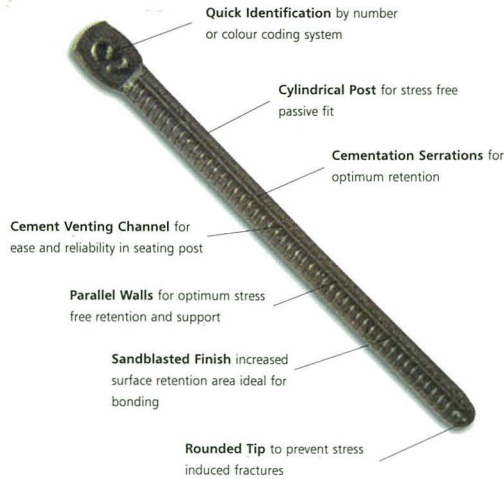

op-po system

Individual Component Parts



Superior Twist Drills

Five diameters of twist drills are available, manufactured in one piece construction of highest quality stainless steel giving optimum concentricity to the user and are fully autoclavable. Millimetre calibration markings. *Non cutting tip reduces risk of lateral perforation.*



Temporary Posts

When impressions are at the laboratory for final fabrication, OP-PO Stainless Steel Temporary Posts can be fitted. The posts are friction fitting but also smooth sided for easy removal with the size identification rings at coronal end which aid retention of temporary crown.



Titanium & Stainless Steel

For direct build up with composite, glass ionomer or amalgam. The OP-PO Titanium Post is manufactured in titanium offering total biocompatibility and corrosion free product. OP-PO Stainless Steel posts are manufactured in Type 304-515.



Precious Metal Posts

For situations which indicate a wrought gold formed post for either direct or indirect procedure the OP-PO Precious Alloy Cast On Post is available (60% Au, 15% Pd, 24.90% Pt, 0.10% Ir)



Impression Posts

OP-PO Impression Posts are smooth sided for easy removal from canal and cast model. They have retention heads for reliability in impression pick up. These posts may be adjusted to required length by simply shortening with a clipper.



Laboratory Burn Out Posts

Serrated and vented mating plastic burn out posts allows the laboratory to fabricate optimum fitting castings in the most trouble free manner. The conical shaped head of the OP-PO Burn Out Post facilitates easy waxing of core build up.



Wrench for hand reaming procedure

The OP-PO parallel post system offers perfection in root canal preparation to the successful treatment result with either direct or indirect procedures. Optimum results are obtained by compatible mating component parts (twist drills, impression posts, laboratory burn out posts, temporary posts, titanium, stainless steel and precious metal posts).

assortment of organiser kits available

For Indirect Technique

Art No OPC6

Product

5 Drills
30 Impression Posts
30 Laboratory Burn Out Posts
30 S/S Temporary Posts

No.

Sizes: 1, 2, 3, 4, 5
6 each Sizes: 1, 2, 3, 4, 5
6 each Sizes: 1, 2, 3, 4, 5
6 each Sizes: 1, 2, 3, 4, 5

For direct build up with Titanium

Art No OPC7

Product

5 Drills
25 Titanium Posts

No.

Sizes: 1, 2, 3, 4, 5
5 each Sizes: 1, 2, 3, 4, 5

For direct build up in Stainless Steel

Art No OPC9

Product

5 Drills
25 S/S Posts

No.

Sizes: 1, 2, 3, 4, 5
5 each Sizes: 1, 2, 3, 4, 5

op-po system re-fill components

Description	White	Grey	Green	Pink	Beige	Blue
Diameter (mm)	0.9mm	1.0mm	1.14mm	1.25mm	1.50mm	1.75mm
Twist Drills	OPD1	OPD2	OPD2.5	OPD3	OPD4	OPD5
S. Steel Posts	OPS1	OPS2	OPS2.5	OPS3	OPS4	OPS5
Titanium Posts	OPT1	OPT2	OPT2.5	OPT3	OPT4	OPT5
Impression Posts	OPI1	OPI2	OPI2.5	OPI3	OPI4	OPI5
Burn Out Posts	OPB1	OPB2	OPB2.5	OPB3	OPB4	OPB5
Temporary Posts	OPTE1	OPTE2	OPTE2.5	OPTE3	OPTE4	OPTE5
Precious Posts	OPP1	OPP2	-	OPP3	OPP4	OPP5

To order now or for further information contact Kerr on (02) 98707500

KERR

Australia PTY
Unit 11c
112 - 118 Talavern Road
North Ryde
NSW 2113

Telephone
(02) 98707500

Facsimile
(02) 98707600

optident
dental solutions